# *Bathmaster\* Xtra*

*User's Manual  
Gebruiksaanwijzing  
Manuel d'utilisation  
Gebrauchsanweisung  
Manuale per l'utente  
Manual del usuario*

**bathmaster**  
X T R A

# The Bathmaster\* Xtra Reclining Bath Lift

## Do the following

- Assess suitability of the unit to lift the individual. Judging the suitability of any patient lifting device for the individual circumstances is the responsibility of the healthcare professional or caregiver. The circumstances that must be considered include the following: size and weight of patient; his or her upper body strength, balance, coordination, cognition, and hip flexibility; and the possibility of spasm.
- Read these instructions completely before attempting to use the Bathmaster Xtra. Become familiar with the operation of the unit before using it to lower or raise a person.
- Keep these instructions where people using the unit can have access to them.
- Inspect the unit before each use to be sure it is free from damage and excessive wear. Annual inspections by trained technicians are recommended.
- Ensure that the feet of the unit and bath surface are completely clean, dry and free from oil, foam and soap residue.
- Ensure that all four rubber feet are securely in place before placing the unit in the bath.
- Ensure that the drive mechanism is securely attached to the base before operating the unit.
- Always use a cover on the unit.
- Adjust the reclining mechanism through the seat opening, and not from behind the seat.
- Have the unit in the lowest position before reclining the back.
- Ensure that the connector is completely dry before plugging it into the recharger or the unit. (Operation and safety of the unit are not affected if the hand controller cable or connector falls into the water.)
- During operation, be sure that the seat, back, and side flaps can move freely, without obstruction. Be sure the user's feet are in front of the seat and the hands are away from the side flaps and the seat whilst lowering and raising the unit. Also be sure the cables will not be tangled or pulled by movement of the unit.
- In case of an emergency, stop operation of the unit immediately by pulling apart the hand controller connector.
- If storing the unit for more than one week, fully charge the battery pack prior to storage and again before the next use. Ensure that the unit is completely dry before storing it.
- Lift the unit from the base, without holding onto the drive mechanism.
- Use only with accessories, hand controller, and rechargers that are recommended by Homecraft AbilityOne.

## Do not do the following

- Do not exceed the lifting capacity of the unit—142 kg (22 stone, 312 lb.).
- Do not use the unit as a support whilst standing.
- Do not use the unit on an incline of greater than 5°.
- Do not use the unit if metal parts of the connectors or cables are exposed.
- If the unit is disassembled, do not operate the hand controller until the backrest and drive mechanism are attached to the base.
- Do not use the unit after the low-battery light on the hand controller has illuminated. Recharge it before use.
- Do not recharge the battery in the bathroom or near any source of water.
- When sitting on the seat of the Bathmaster Xtra do not push against the bath, such that the front of the bath lift is raised off the floor of the bath, as this will make the bath lift unstable.

## General use

The Bathmaster Xtra is a patient bathing device intended to help people get into and out of a bath. This fully powered, self-contained bath seat requires no plumbing or electrical installation.

The unit, which stands on two sucker feet and two dimpled feet, is easily positioned in most standard baths. It folds compactly, providing convenient carrying and storage. The backrest may be removed for easier fitting into and removal from the bath.

Replacement components and accessories are available.

## Standard components

The Bathmaster Xtra includes the following components:

- Stainless steel and polypropylene unit
- Removable seat cover
- Hand controller containing battery pack with LED low-battery warning light

- Battery recharger
- Side flap protectors
- User instructions

## Charging the battery

The hand controller must be charged prior to first use and whenever it is low on power. Charging the battery should be done away from the bathroom and other sources of water.

The light on the battery pack illuminates when the charge is low. When the light goes on, enough power remains for one complete lift. However, the unit will not lower into the bath.

To charge the battery, disconnect the cable of the hand controller from the unit. Ensure that the connector is dry, connect the cable of the battery pack to the cable of the recharger, and plug the recharger into an electrical outlet. The light on the recharger illuminates when the hand controller and recharger are connected correctly, and it remains illuminated even when the battery is fully charged (on European rechargers, the light pulses when charging is complete).

A complete charge takes approximately 15 hours; the battery can be left continuously connected to the recharger. It is recommended that the hand controller be placed on charge following each use and left on continuous charge so it is ready when needed.

## Improving battery performance


If the capacity of the hand controller has noticeably reduced (i.e., the unit provides fewer lifts than expected per full charge), it may need reconditioning. To recover the full capacity of the battery, use the following procedure:

1. Remove the Bathmaster Xtra from the bath.
2. Operate a cycle once or twice with a person on the seat.
3. Operate the unit repeatedly up and down, without a person on the seat until the motor stops of its own accord (other than at the top or bottom limits of travel).
4. Recharge the hand controller for 15 hours.
5. Repeat steps 2 through 4 four more times.

## Preparing the unit for use

1. Check the hand controller to ensure that there is sufficient power to complete a full cycle. The LED low-battery warning light on the hand controller illuminates to indicate that enough power remains for one complete lift only. If necessary, recharge the battery prior to use.
2. Remove the backrest from the base by tilting the backrest forward until the hinge is aligned with the slot in the base hinge and then lifting it off the unit. Apply the seat cover as follows:
  - a. Hook the front of the seat cover over the front of the seat of the unit.
  - b. Raise the side flaps and slip the cover over them, one at a time.
  - c. Pull the tabs on the back of the seat cover over the backrest hinges.
  - d. Slip the top of the backrest cover over the top of the backrest and loop the elasticated bottom around the lower end of the backrest.


Notice the opening in the seat cover that gives access to the metal release (labelled **A** in the photo), allowing the back to recline or fold onto the unit.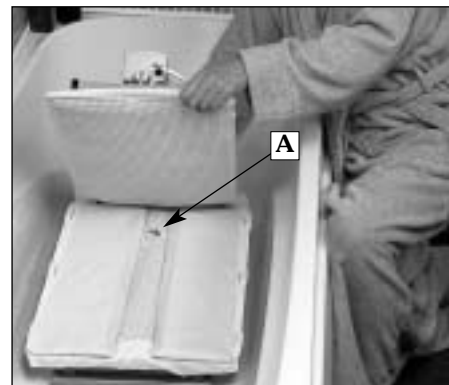

3. The side flap protectors enable the side flaps to move smoothly up and down without catching beneath the handles on the bath. If they are needed, clip the protectors into place on the handles with the short

edge toward the outside of the bath. When fitted correctly, they should rest at the bottom of the edge of the bath. If necessary, they may be trimmed to length.

- Place the base of the unit into the bath as far away from the taps as possible. If the taps are at one end of the bath, place the unit so it faces them. Allow room at the back of the bath for the back of the unit to recline.
- Push both sucker feet securely onto the bottom of the bath.
- To fit the backrest onto the seat unit, angle the backrest forward until the hinges are aligned with the slots in the seat brackets. Simply lower the hinges into the brackets, ensuring that they locate fully, and push the backrest backwards until it clicks into the upright position.
- The seat has four positions between upright and full recline. To recline the back, ensure that the seat is at the lowest level. Press firmly down against the metal release through the seat whilst holding the backrest and clicking it into the desired position. To ensure that it is locked in place, hold the top of the backrest and apply pressure backwards and forwards.
- Attach the hand controller by pushing together the connectors on the drive mechanism cable and the hand controller. Making sure the cables are away from the unit, check that the unit is able to move up and down without obstructions by using the hand controller to raise and lower the seat.
- To lower the unit into the bath, depress the down pad (labelled **B**). Press the up pad (labelled **C**) to raise the unit. To stop the unit at any time, simply remove pressure from either pad. The pads have a large surface area to enable them to be operated using any part of the hand.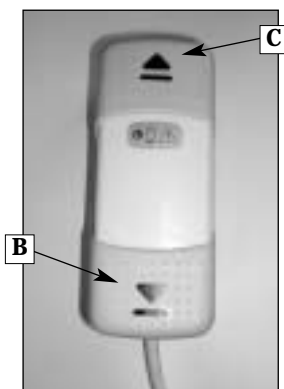

### Spare sucker feet

Your Bathmaster Xtra has been supplied and fitted with a pair of flat dimpled feet on the rear and a pair of suction cup feet on the front. The combination of feet has been successfully tested for stability. However, fitting suction cup feet to the rear as well as the front has been found to offer greater security in certain deeply sloping or contoured baths. For this reason, we have included a second pair of suction cup feet for you to fit to the rear of your Bathmaster Xtra if required.

The suction cup feet have a wire clip arrangement that ensures the feet cannot be inadvertently pulled off of the ball joints on the frame. To fix the feet onto the frame, prise the wire clip out of the sucker foot using a small screwdriver or penknife if necessary. Push the foot onto the ball joint and replace the wire clip. Ensure that the clip is replaced via the flat side of the moulding and that the clip is pushed fully home.

### Using the unit whilst bathing

**In case of an emergency, operation of the unit can be stopped immediately by disconnecting the hand controller from the unit.**

- If possible, the user should operate the hand controller so movement can be stopped instantly in case of discomfort. Releasing the switch at any time stops the movement of the unit. The unit stops automatically when it reaches its lowest and highest levels.
- With the seat level with the rim of the bath, check the unit for stability to ensure a secure transfer onto the seat.
- Position the user on the seat and swivel the body to swing the legs into the bath.
- Ensure that the user's feet are in front of the seat, not tucked under it, and the hands, cables, and all other objects are away from the side flaps and the seat.
- After bathing is complete, raise the unit until the seat is level with the rim of the bath so the user can get out of the bath.

### Removing the unit from the bath


To reduce the weight of the unit, and therefore the possibility of back injury, disassemble the unit prior to removing it from the bath. It is also

possible to remove the unit from the bath folded.

For ease of transportation and for storage, the unit can be folded or disassembled as follows:

#### Removing the unit disassembled

- Allow any water to drain from the bath and fold the side flaps onto the seat.
- With the seat in its lowest position, use the opening in the seat to release the back, tilt the back forward until the hinge is aligned with the slots in the base hinge (labelled **D**), and lift the backrest off the unit.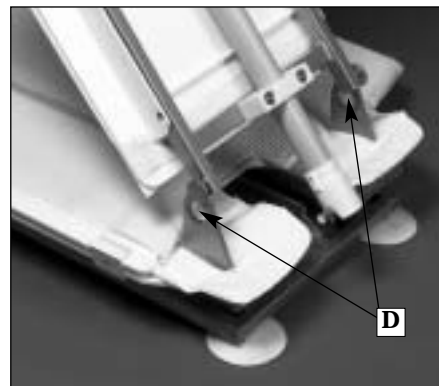

- Pry up the tabs on the sucker feet whilst tilting the unit sideways and then backwards so the feet do not become re-attached.
- Remove the base from the bath.

#### Removing the unit folded

- Allow any water to drain from the bath and fold the side flaps onto the seat.
- With the seat in its lowest position, use the opening in the seat to release the back and tilt it forward until it lies on the seat.
- Pry up the tabs on the sucker feet whilst tilting the unit sideways and then backwards so the feet do not become re-attached.
- Remove the unit from the bath.

### Care and cleaning

The unit should be cleaned weekly, as follows:

- Clean the backrest and base portions of the unit with warm water and mild detergent or disinfectant, using a brush or cloth. Avoid using strong disinfectants. Rinse unit thoroughly and dry.
- The base portion may be autoclaved or cleaned with a high-pressure hose if required.
- The seat cover is recommended for use by only one person. It can be wiped with a damp cloth or hand washed using warm water and mild detergent. A brush may help with cleaning; however, the abrasion will reduce the life of the seat cover.

### Warranty

Homecraft AbilityOne warrants that the Bathmaster Xtra Reclining Bath Lift is free from defects in material and workmanship. This warranty shall remain in effect for the following terms:

- 2 years from the date of original consumer purchase for the frame and motor
- 1 year from the date of original consumer purchase for the hand controller and recharger

In the unlikely event that the unit needs service, contact Homecraft AbilityOne or the distributor for instructions and repair.

This warranty does not cover the following:

- Replacement parts or labour furnished by anyone other than Homecraft AbilityOne, the distributor, or any approved service agent.
- Any failure of the unit during the warranty period if the failure is not caused by a defect in material or workmanship or if the failure is caused by unreasonable use.

**Homecraft AbilityOne is not liable for incidental or consequential injury or damage to property.**

### Servicing the unit

If the product requires service or repair, contact the retailer from which it was purchased or Homecraft AbilityOne on +44 (0)1623 754047.

## Replacement components and accessories

### Replacement Feet

Sucker feet and dimpled feet with nodules on the underside hold the unit securely in the bath.

Sucker feet (set of 2) **AA139001**  
 Dimpled feet (set of 2) **AA139002**

### Seat Covers

Seat covers, which increase user comfort, are intended for use by only one person. Replacement seat covers are available singly and in packages of 10.

pkg. of 1 **AA1396**  
 pkg. of 10 **AA1397**

### Hand Controller with Battery Pack

This hand controller has a low-battery warning light. When lifting a person of average weight, a fully charged battery provides eight full lifts per charge. For use with the battery recharger below.

**AA1395**

### Battery Recharger

For use with the hand controller above.

North America **AA1337U**  
 Europe **AA1337E**  
 Australia / New Zealand **AA1337A**  
 UK / Ireland **AA1337**

### Swivelling Transfer Seat

Circular transfer seat glides laterally to facilitate smooth transfers onto and off of the unit. It also rotates to help swing the user's legs into the bath.

**AA1392**

### Side Flap Protectors

Allow smooth gliding of side flaps over handrails in a bath.

**AA1349**

## Technical specifications

- Degree of protection against ingress of water:
  - Actuator motor housing . . . . .IP66
  - Hand Controller . . . . .IP67
- Motor power rating: . . . . .12V  
 Max 8 amps
- Transport and storage conditions:
  - Temperature . . . . .-5 deg C to +40 deg C (-20 deg F to +104 F)
  - Relative humidity . . . . .30% to 70%
  - Atmospheric pressure . . . . .700Pa to 1100hPa
- Conditions for use:
  - Temperature . . . . .-5 deg C to +50 deg C (-20 deg F to +125 F)
  - Relative humidity . . . . .30% to 100%
  - Atmospheric pressure . . . . .700Pa to 1100hPa
- Type of protection against shock: . . . . .Internally powered equipment
- Degree of protection against electric shock: . . . . .Type B
- The mode of operation is 10% (1 minute on, 9 minutes off)
- The bath seat is made from stainless steel and plastic, please separate and dispose of correctly.
- Total product weight . . . . .12.4kg
  - Disassembled:
  - Base frame assembly . . . . .8.3kg
  - Backrest frame assembly . . . . .4.1kg


### Key guide

See page 20.

### Troubleshooting guide

If the Bathmaster Xtra fails to operate, check the following:

Possible cause	Solution
Is the correct button on the hand control being pressed?	Arrows on the buttons point away from or towards the 'water' to indicate the direction of travel as shown at right.
Is the hand control fully connected to the actuator cable?	Push the connectors together until the rubber seal meets the female socket.
Does the hand control have sufficient charge?	See <b>Charging the battery</b> for instructions.

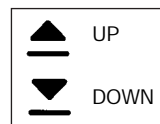In the unlikely event that problems other than those listed above are encountered, contact the retailer from whom the unit was purchased or Homecraft AbilityOne on +44 (0)1623 754047.

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